biolight  <sup>TM</sup>

biolight  <sup>TM</sup>

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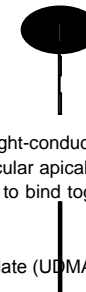
Marquage CE : 2013

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### Description

Biolight ST and Biolight Dual are radiopaque and light-conducting dental post, made of a glass fiber-reinforced composite. Biolight ST has a Mooser-type conically radicular apical part whereas Biolight Dual has a high double conical radicular apical part. The use of an adhesive technique to bind together the post, the core material and the root dentin shows long term favorable outcome.

The polymer matrix is composed of urethane dimethacrylate (UDMA) and doesn't contain BisGMA.

### Indication

Endodontically treated teeth post and core build-up restorations. Various sizes are available to cover all practitioners' needs (the appropriate size will be chosen by means of a radiographic image).

#### Biolight ST

Size	1.0	1.2	1.35	1.5	1.7	2.0
Packaging color	Red	Blue	Green	Black	White	Yellow
Length (mm)	19	19	19	19	19	19
Coronal diameter (mm)	1.00	1.20	1.35	1.50	1.70	2.00
Apical diameter (mm)	0.52	0.65	0.72	0.82	0.92	1.02

#### Biolight Dual

Size	#0.5	#1	#2	#3
Packaging color	Black	White	Yellow	Blue
Length (mm)	20	20	20	20
Coronal diameter (mm)	1.25	1.50	1.80	2.20
Apical diameter (mm)	0.80	0.90	1.00	1.20

Biolight ST and Biolight Dual post are available with corresponding drills (see color code).

### Contra-indications

- Allergy to methacrylates.
- Less than 2-3 mm superingival tooth structure.

### Side effects

At the present state of knowledge, no side effect has been reported.

### Precautions

- The dental post must be cleaned only with medical alcohol prior to insertion in the canal.
- Avoid direct contact with fingers.
- The adjustment of the dental post to the appropriate length must be done outside the mouth.
- The coronal adjustment should be done under water spray and a surgical suction device should be used. We recommend the use of a dental dam. Protective glasses, surgical mask and gloves should be worn.
- In case of irritation the discomfort can be eased with water and mild soap (mechanical action).
- Dental post is a single-use device and must not be reused.

### Biolight ST/ Dual Instructions for use

- 1) Remove gutta percha from the canal. Leave a 5 mm apical seal.
- 2) Choose the Biolight ST/Dual size corresponding to the tooth to be treated.
- 3) Drill the canal with the drill corresponding to the post size - according to needs (1000-2000 rotations/ min).

- 4) Clean the post with alcohol and control that the position in the mouth is correct.
- 5) X-rays control.
- 6) Adjust the coronal extend of the post. Cut out excess portion outside the mouth:
  - Use either a diamond bur with a water spray or a low gear carborundum disc without water.
  - Do not use cutting-off pliers.
  - Do not hold tight with pliers: delamination risks.
- 7) Clean and dry the canal with air and paper points (remains of eugenol can inhibit the polymerization of most bonding materials).
- 8) The post must be cleaned with alcohol only, before definitive implementation.
- 9) Post bonding using an adhesive process:
  - Coat the post with a silane for composites, then let it dry.
  - Etch the walls of the canal (37% phosphoric acid) and rinse thoroughly with water.
  - Use an adhesive primer for composite materials. We advise not to use self-etching acid adhesive primers (acid may inhibit the polymerization of the bonding material).
  - We advise to use self-curable composites only or resin modified ionomer glass cement.
  - We advise to use a dual curable bonding material meant for composite posts or a self-curable bonding material (to the exclusion of the 100% photo bonding material).
  - Fill the canal with cement using a root canal filler to insure proper coating of the canal and place the post to proper length. Apply pressure with a gloved finger.
  - Light cure or allow the cement to set undisturbed for 10 minutes.
- 10) Core build-up:
  - Process according to the professional practice for composite filling materials.

**Cleaning, disinfection and sterilization** (Endodontic posts and drills are not delivered sterile)

- Clean and autoclave (134°C for 18 min.) drill before each use. Use a separate autoclave bag for each instrument.
- Use a suitable agent for cleaning and disinfecting rotating instruments (in accordance with EN 1040 and EN 1275). Follow the respective manufacturer's instructions. Some aldehyde-free cleaning or disinfecting solutions can cause discoloration of the color-coded rings. Use a brush or ultrasonic bath (without contact to other instruments) for cleaning. Rinse well with water and dry afterwards. Verify that cleaning has been thorough and repeat if necessary
- Always store drills dry and in a dust-free container. Visually check the drill before each use. Incompletely cleaned drills have to be cleaned again. Separate out damaged and/or dull drills and discontinue using them.
- The posts can be disinfected with medical alcohol.

**Guarantee – Limited liability**

The guarantee is limited to the quality of the product: any defective product will be replaced. The Batch Number must be cited in all correspondence which asks for identification of the product.

This product has been developed for use in dentistry and must always be used according to the Directions. Any harm resulting from a failure to comply with these provisions, incorrect handling or utilisation for purposes other than those indicated in the Directions, will not be the manufacturer's responsibility. Before every use, the user is obliged to check the compatibility of the equipment with the intended application: in consequence, he or she is entirely liable for use of the product and any associated damages.

**Note**

Product reserved for use in dental applications.

Keep out of the reach of children.

Biolight ST and Biolight Dual are made in France.

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